EMA-LED GmbH

Ottostrasse 3

63785 Obernburg am Main

Germany

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OPERATIONSLEUCHE

EMALED 560

oder

EMALED 500

OPERATING MANUAL

SURGICAL LIGHT

EMALED 560

or

EMALED 500

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This operating manual provides you with the necessary information for commissioning and reliable operation of your EMALED surgical light. Therefore read the operating manual before the commissioning.

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This operating manual is intended for trained staff. The contents of this manual must be available to the staff and implemented by the staff.

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The surgical lights **EMALED** are intended for surgical site lighting by surgical operations, diagnostic study and medical examinations in hospitals. The surgical lights are intended for ceiling mounting. The control of the surgical lights are carried out by the control units of the lamp body. With the help of the control unit, the operating modes of the surgical lights as light output and work area size can be adjusted.

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This symbol means helpful additional information

Caution: Non-observance of this warning may lead to failures or malfunction.

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Warning: Non-observance of this warning may result in danger to persons and/or serious damage of the device.

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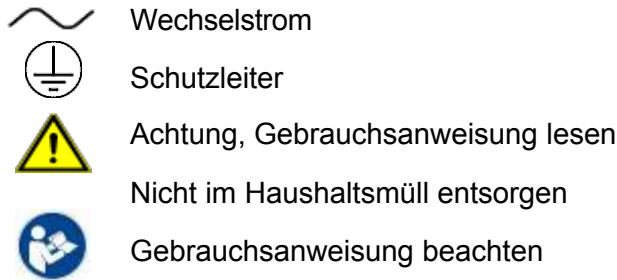
Hazard: Non-observance of this warning may result in serious injury of persons and/or destruction of the device.

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List: The point in front means a list without strict order.

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The following symbols are used on the surgical light:

Alternating current
Protective earth ground
Attention, read manuals
Do not dispose in household garbage
Follow operating instructions

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All the operations described in this manual must be performed only by trained and authorized staff. Operations beyond this must be performed only by authorized staff due to safety and warranty reasons.

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The surgical lights **EMALED** are intended for lighting of operative and examination field. You can find detailed information about use of surgical lights EMALED in chapter 1.3 „Destination“ and in Chapter „Product description“.

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By misuse or unintended use some dangers may come from these devices, e.g. glaring of the staff or device failures because of incorrect mounting.

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The surgical lights EMALED are matching to the technical level by observance of applicable standards and regulations. The user has to respect safety rules in the operating manual, national mounting standards as well as safety regulations and accident prevention rules.

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Surgical lights are CE conform to MDD 93/42 EGW.

The device meets the electromagnetic compatibility requirements according to:

- IEC 60601-1-2: 2014, EN 60601-1-2: 2015 (german version)
- IEC 60601-1: 2013-12, clause 4.10.2

The device meets safety requirements for surgical lights according to

- IEC 60601-1: 2013, EN 60601-1: 2006 + A1: 2013 (german version)
- IEC 60601-2-41: 2016-02, EN 60601-2-41:2009 + A1: 2015 (german version)

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The surgical lights are classified in accordance with Directive 93/42/EWG of the June, 14th, 1993 on medical products to the rule 12, class I (not a product of class II a), Appendix VII.

The user must make the intended use of the device and observe the information in the following documentations:

- This operating manual
- This declaration of the manufacturer
- The relevant mounting regulations

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The protection of natural resources is one of the most urgent tasks. Therefore we introduced an environment management system in order to continuously improve the environmental protection at the place.

Help us to meet these requirements and observe the environmental regulations in this operating manual:

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Delivery scope:

Delivery scope consists of:

- Surgical light EMALED 560 or EMALED 500
- Documentation
- Operating manual

Components:

The surgical light consists of following components:

- Ceiling mount with cover
- Suspension tube
- Suspension system «ACROBAT 2000»
- Light head
- Sterilisable handle
- Railing of the lighthouse

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General view of the light

1. Ceiling mount with cover
2. Suspension tube
3. Central axis «ACROBAT 2000» with spring arms
4. Lighthead
5. Sterilisable handle
6. Railing of the lighthead

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Application field

The surgical lights **EMALED** are designed for lighting of the operative and examination field. The surgical lights EMALED 560 or EMALED 500 contains light sources. As light sources light-emitting diodes (LED) are used. The position of LEDs is harmonized with the optical system of the light head. The light head emits qualitative light which has a natural color (estimated in color temperature) and color rendition similar to day light. Due to higher illuminance and better color rendition it is possible to recognize even the smallest particles and changes in color of tissues and organs. Reflexion and lences technology is used.

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Principle of operation

The whole system of a surgical light EMALED 560 or EMALED 500 consists of one light head, which is mounted by the suspension system „ACROBAT 2000“, the suspension tube and ceiling mounting set. The suspension system „ACROBAT 2000“ with spring arms enables unlimited rotatory motions of the light head and its positioning at any height and at any position.

Power supply

Normally the surgical lights are powered by the power supply unit (power supply unit is in the delivery set) with secondary voltage (24 V) The primary side of the system is powered by alternating-current mains (230 V, 50-60 Hz). When equipped with back-up batteries (optional) surgical lights switch automatically to emergency operation. During emergency operation the surgical lights are powered by integrated back-up batteries. Parameters are shown in chapter 9.1 „Specification of the surgical lights“.

Battery charging (optional)

As long as the batteries are connected to a charging unit the batteries are charging permanently. A green LED PWR indicates a mains operation of the whole device and thereby the charging of batteries. By this reason we abandoned consciously visual status indication.

Pay attention to symbols used on the device.

Alternating current

Protective earth ground

Attention, read manuals

Off (Power supply: disconnected from the circuit)

On (Power supply: connected to the circuit)

3.3 Bedienung / Control

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See chapter 6.3 „Preparation and execution of work“

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Package

Your device is protected on the way to the place of consignment by a package.

Thereby the usual transport encroachment are tested according to DIN 55439.

The standard devices are protected by cardboard wrapper, it is environment friendly and recyclable. Light heads are additionally protected by PE film and polystyrene foam.

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Dispose of the packaging material with specialized recycling companies.

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Temperatures during storage and transport

- Relative air humidity during storage up to 80% at +25 °C.
- Storage temperature from + 5°C to + 40°C.
- Atmospheric pressure during storage from 840 to 1067 hPa (from 630 to 800 mm of mercury column).
- Transport temperature from - 25 to + 45 °C must not be exceeded.
- Transport time under stated conditions must not exceed 4 months.
- The device is to handle with care and not to expose it to excessive vibration and jolting.

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The surgical light may be mounted only by EMA-LED GmbH or by EMA-LED GmbH authorized or specialized company.

Before mounting, the customer has to provide written evidence of load - bearing capacity of the ceiling for one or another anchorage type of surgical lights.
(Pre-printed form is enclosed to the Installation instructions).

The surgical lights must be mounted and assembled according to **“Installation instructions“**

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It is a mandatory for the user to provide for disconnecting device from mains installed in supply line to surgical lights

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Follow the safety instructions

It is obligatory to respect the following safety instructions:

- The device must be connected only in de-energized state.

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The device must be connected according to the **„Installation instruction“**

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Upon completion of installation works an acceptance certificate for surgical lights and controller device is to prepare and sign together with the user. (The pre-printed form is enclosed to the Installation manual).

The surgical light is ready for operation as soon as the personal has been trained in using and every person has confirmed with signature.

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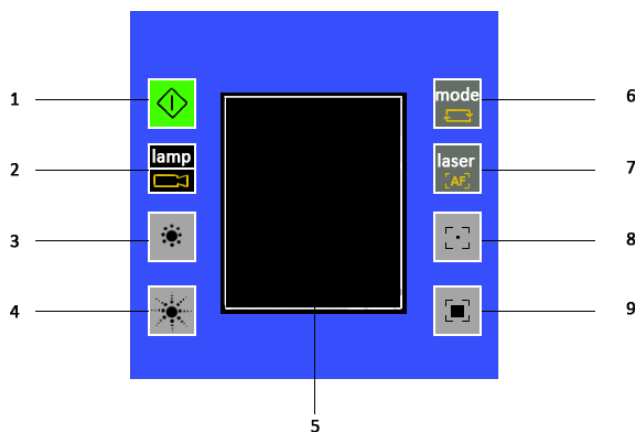
The surgical lights may be used only under climatic conditions, as stated in chapter 9.1 of this operating manual.

Additional weight loading of the structural system is not allowed

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- 1 On / Off switch
- 2 switch light / camera
Only active with optional camera
- 3 illuminance -
In camera mode Zoom -
- 4 illuminance +
In camera mode Zoom +
- 5 Display
- 6 Mode switch
 - Endo mode
 - Central mode
 - Normal mode
- 7 laser pointer
In camera mode Autofocus On/ Off
- 8 Focus -
- 9 Focus +

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Turn on

Pressing the button no. 1, the operation light is turned on.

Turn off

Long press on the button no. 1, the operating light is switched off.

Regulation of illuminance

By pressing the buttons no. 3 and 4 you can adjust the light intensity as desired.

Setting the size of the working field

By pressing the buttons no. 8 and 9 you can set the size of the working area to the desired size.

Turn on of the optional Laser Pointer

The laser pointer marks the center of the camera and is used to align the optional camera on the surgical field. By pressing the button no. 7 you can turn on the laser pointer – it switches off again after 30 sec.

Switch light / camera

The button no. 2 enables you to control the optional camera. For more detailed information please read the manual of the camera.

Mode switch

Pressing the button no. 6 enables you to switch between the different optional operating modes.

Standard mode

The lamp can be operated via the control buttons on the light head

Endo mode

The light turns on to the maximum lighting field and about 15% of max. Illuminance and thus enables the orientation in the room as intervened in endoscopic.

Central mode

The light adjusts to the smallest lighting field and to 100% of illumination, enabling depth illumination of the operating field

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Display

On the display (no. 5) the current mode is displayed.

In standard mode, the illumination and the size of the focus is displayed in the bar graphs

Standard Features

- No movable parts in the light head
- Adjustable illumination in 10 steps
- Powder coated aluminium case
- Two autoclavable handles are included additional
- High flexibility: 360° rotation horizontal
- 4-step focusing via the control panel with LCD display
- Endoscopic mode
- Macro mode

Optional extensions EMALED 560 or EMALED 500

- Full HD camera in the center of the surgical light
- Colour temperature change: 3500, 4000, 4500 K (optional)
- Laser pointer (optional)
- Remote control
- Emergency power back-up (for up to 4hrs network independent use)

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The surgical light EMALED 560 or EMALED 500 is maintenance-free, an annual maintenance is recommended by an authorized service center.

Before performing maintenance work the light head has to cool down!

To replace the sterilizable handle the ball snap must be pressed on the handle, then the handle can be pulled down.

When putting the handle back on, make sure that the ball catch is locked and the handle has tightened.

The removable, sterilizable handles are sterilized with steam.

For sterilization, the handles can be put vertically with the opening upwards in the holder of the device.

Sterilization temperature should not exceed 134°C. During sterilization you have to avoid contact with other objects.

Replace defective handles (optical crack testing), because particles can get from defective handles in the patient's wound.

In compliance with the above rules, the handles are designed for min. 350 sterilization cycles.

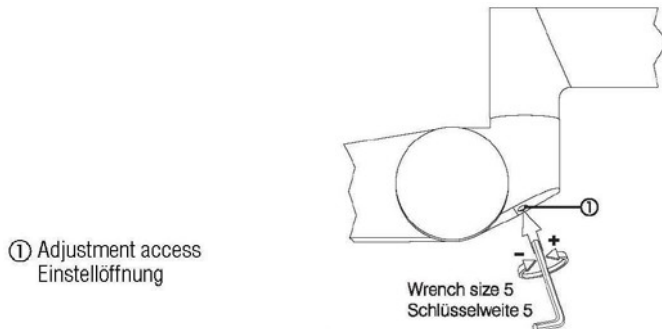
Perform hygienic treatment (disinfection) of the luminaires exclusively with 3% hydrogen peroxide (H₂O₂) in addition of a weak soapy detergent solution. Perform disinfection of the lights only when cooled down.

Do not use abrasive, alkaline, acid and alcohol based cleaning disinfectant, to prevent damage of plastic parts.

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The adjustment and care work must be carried out by a hospital technician (or a person with comparable qualifications).

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To adjust the spring force:

make sure that the spring arm with the end device can come to rest in any desired position.

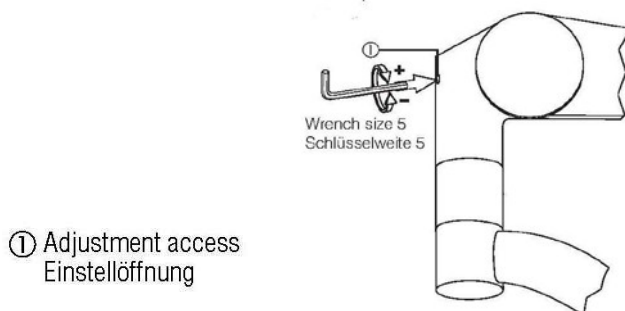
1. Insert Allen wrench (size 5) into adjustment access (no. 1).

If the spring arm drops, the spring force is too low:

- rotate Allen screw in the counterclockwise (+) direction.

If the spring arm rises, the spring force is too high:

- rotate Allen screw in the clockwise (-) direction

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When setting the vertical limit, ensure:

that collisions with other components are excluded (the vertical stop can be limited to the horizontal level).

1. Insert Allen wrench (size 5) into the adjustment access (1)

To reduce the vertical lift:

- rotate Allen screw in the clockwise (-) direction.

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To increase the vertical lift:

- rotate Allen screw in the counter clockwise (+) direction.

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The accumulators for the emergency mode must be reviewed annually by the EMA-LED or a specialist company, accredited and authorized by EMA-LED GmbH

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The batteries used are maintenance-free!

You can find the batteries for the OP-lights under the cover of the ceiling fixture.

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We recommend to replace batteries every 2 years by a specialized company.

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Replace all batteries simultaneously. Make sure that the accumulators are of the same type and from the same manufacturer.

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The first measure is to check the power supply. A cold start of the system can be performed, too. In many cases, the causes can be observed in this way and faults rectified.

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Repairs to the OP light may generally only be performed by the EMA-LED GmbH or a specialized company.

Only use spare parts and accessories of the manufacturer for maintenance and repair.

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Before starting any repair work, the OP lights have to be de-energized.

Before starting any repair work, please let the OP lights cool down min. 20 minutes.
Replace damaged parts immediately!

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Before making any remove at the OP lamps on mains cut-off (mains isolation switch). The equipment must be de-energized always!

Perform the steps of the assembly instructions in reverse.

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The OP lights are made of materials that can be recycled by specialized recycling companies. We have designed the electronic modules to be easily separable and used reusable materials.

Pass the instrument directly on to a specialized recycling company and do not use for the municipal garbage. This is allowed only for privately used products according to the WEEE - directive

Proper disposal prevents negative effects on human health and environment and ensures recycling of useful raw materials.

MATERIALS:

Metal parts: housing of OP lights with accessories, structural systems.

Plastic parts: protective glass railing of the lighting fixture, sterilizable handle and its holder, cover of the lighting fixture.

Electrical products: LED modules, power supply, charger, terminal blocks, PCB of filament.

Accumulators.

If you have no way to dispose of the instrument properly, please speak with us about return and disposal.

**Do not throw batteries in the trash!
Dispose used batteries according to manufacturer specifications.**

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Provided that the transport, storage, mounting and operation conditions are met the manufacturer guarantees the conformity with specifications.

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Warranty period is 24 months from the date of commissioning of the surgical lights with the controller device.
The warranty does not apply to consumable and wearing parts.

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Warranty period for storing surgical lights and controller devices is 6 months

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Please note in the table any failures and disturbances of the OP lights and the operation unit that occurred during the operation as well as corrective actions.

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EMA – LED GmbH